## EN About Cohesive® skin barriers

**What's this product**  
eakin **Cohesive®** skin barriers are moisture absorbing skin protecting barriers which protect skin from harmful attack by aggressive body fluids. High tackiness on both sides ensures that any subsequent appliance is held securely in position on the body.

eakin **Cohesive®** skin barriers can be moulded, rolled, stretched or compressed to any shape to accommodate irregularities in the skin around stomas, wounds and surgical drains.

**Indications for use:** eakin **Cohesive®** skin barriers are intended to adhere to peristomal or around skin and can be used as:

- A sealing agent around wounds and surgical drains
- An external packing agent on uneven skin
- An ostomy seal

### Instructions for use

- Application**
1. Ensure that the area for application is clean, dry and free from traces of grease or cream.
  2. Remove the eakin **Cohesive®** skin barrier from its packaging and warm between your hands for a few seconds before removing the backing paper.
  3. Gently stretch the eakin **Cohesive®** skin barrier as required to fit the application site. It may also be cut and re-joined as necessary to fit around more difficult sites.
  4. Apply the eakin **Cohesive®** skin barrier to the skin with gentle pressure and hold it in place for a short time until it is fitted securely. Overlap the two ends of the product (if cut) for complete skin protection.
  5. Mould the eakin **Cohesive®** skin barrier towards the edge of the wound or drainage site to ensure a tight seal and provide maximum skin protection.
  6. Place any appliance directly onto the surface of the eakin **Cohesive®** skin barrier. Hold the appliance in place with gentle pressure until securely attached to the eakin **Cohesive®** skin barrier and the surrounding skin.

## FR Removal

Gently remove any appliance from the body.

Remove the eakin **Cohesive®** skin barrier by carefully peeling and rolling from the outer edge.

Remove any excess by gently rolling and peeling.

A medical adhesive remover can be used to remove residue from the body or the appliance.

### Disposal

Dispose of waste products in accordance with local guidelines.

### Warnings and cautions

Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience any skin irritation, redness or discomfort in the area of product application, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds. Do not re-use. Re-use of a single use product may have potential to cause harm to the user due to the risk of contamination and impairment in product function.

### Other information

Store flat in the carton at room temperature between 10° and 30° Celsius. Do not refrigerate.

**eakin®** accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to **eakin®**'s current recommendations.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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