EN **eakin freeseal**[®] is a moisture absorbing, skin protecting seal which protects peristomal or periwound skin from harmful attack by aggressive body fluids. High tackiness on both sides ensures that any subsequent appliance is held securely in position on the body.

Indications for use:

- An ostomy seal
- A sealing agent around wounds and surgical drains
- An external packing agent on uneven skin

Intended users

Für ostomates, healthcare professionals and caregivers.

Instructions for use

Application

1. Remove the seal from its outer package.
2. Ensure that the area for application is clean, dry and free from traces of grease or cream.
3. Warm the seal in your hands for 20 – 30 seconds until soft before removing the protective discs. Gently stretch the **eakin freeseal**[®] as required to fit the application site. It may also be broken or cut and re-joined if necessary to fit around difficult sites.
4. Apply to the skin with gentle pressure and ensure

5. Ensure a snug fit with no gaps around the base of the stoma to provide maximum skin protection.
6. Place any appliance directly onto the surface of the **eakin freeseal**[®]. Hold in place with gentle pressure until securely attached to the **eakin freeseal**[®] and the surrounding skin.

Removal

Gently remove any appliance from the body. Remove the **eakin freeseal**[®] by carefully peeling from the outer edge. For ease of use, the majority of the seal should come away in one piece.

Disposal

Dispose of waste products in accordance with local guidelines.

Warnings and cautions

Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience

any of these symptoms, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds.

Other information

Store flat in the carton at room temperature between 10° and 30° Celsius. Do not refrigerate.

eakin[®] accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to **eakin**[®]'s current recommendations.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

EN Expiry date **CZ** Datum expirace **DE** Verfallsdatum **DK** Udløbsdato **EE** Kõlblikusaeg **ES** Fecha de Caducidad **FI** Vähemäinen käyttöpäivä **FR** Date d'expiration **GR** Ημερομηνία λήξης **HR** Rok trajanja **IS** Fyringardagsetning **IT** Data di scadenza **NL** Expiratiedatum **NO** Utløpsdato **PL** Data ważności **RU** Дата истечения срока действия **SE** Använd före **SI** Rok uporabnosti

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