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Wound POUCHES™

A separate large or small access window (839280/839281) is available which can be placed anywhere on the larger pouches to gain access.

All Eakin Wound Pouches are also available with a 8.5 mm diameter bung closure attachment for night drainage if required.

Empty the pouch at regular intervals. Frequent emptying of the pouch prevents 'drag' and prolongs its useful life.

4. Tuck the folded outlet into the pocket to secure the closure.

3. Fold the doubled outlet level with the bottom pocket.

1. Open the outlet.

The pouch may be opened by holding the bottom of the outlet pocket and gently releasing the outlet tube. The outlet may be closed and repositioned as follows:

5. Apply gentle pressure around the skin barrier and hold for at least 30 seconds.

4. Remove the release paper from the skin barrier and place the pouch on the wound.

3. Prepare the patient by cleaning the skin around the site. The Eakin Cohesive® skin barrier of the pouch will adhere to moist skin, however, a better result is obtained when the skin is dry.

2. Align the template to the skin barrier on the pouch and cut the desired shape.

1. Draw the shape of the site on the template provided.

Eakin Wound Pouches are flexible pouches designed to offer skin protection and contain drainage from wounds, fistulae and irregular or multiple stomas.

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