EN

eakin surround™ Skin Protector

About this product

eakin surround™ can be used to protect skin around devices which are inserted and held subcutaneously for a period of time...

eakin surround™ is composed of eakin Cohesive® and a thin, breathable backing film to prevent it sticking to clothing.

Instructions for use: Application

- 1. Ensure that the area for application is clean, dry and free from traces of grease or cream.
2. Remove eakin surround™ from its package and warm between your hands for a few seconds before removing the backing paper.
3. Apply eakin surround™ to skin around the tube, applying gentle pressure.
4. Hold in place for a short time until securely attached.

EN

Other information

Store flat in the carton at room temperature between 10° and 30° Celsius. Do not refrigerate.

eakin™ accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to eakin's current recommendations.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use.

Table with 4 columns: Diameter, Pack Size, Product Code. Rows for Large Skin Protector (98mm, 10, 839011) and Small Skin Protector (58mm, 20, 839012).

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© 2020 TG Eakin Ltd. Issue 1, 2020-01-01
TG Eakin Ltd