EN Medical device **CZ** Zdravotnické prostředky
DE Medizinprodukt **DK** Medicinsk udstyr
FI Lääketieteellinen laite **FR** Dispositif Médical
IS Lækningsgæðunáður **IT** Dispositivo medico
NL Medische loepassing **SE** Medicinsk utrustning
SI Medicinski pripomoček **AE** جهاز طبي

EN Capable of being recycled (carton only)
CZ lze recyklovat (pouze kartony) **DE** Karton kann recycelt werden **DK** Kan genanvendes (kun kartonen) **FI** Vaidaan kierrätettävä (vain pahvi) **FR** Emballage carton recyclable
IS Endurvinnanlegt (eðlens óhjúg) **IT** Riciclabile (solo cartone) **NL** Geschikt voor recycling (verpakking) **SE** Kan återvinnas (endast kartong) **SI** Primerno za reciklazo (samo kartona) **AE** قابل لإعادة التدوير

EN Dry flat **CZ** Sušit na plocho **DE** Flach trocknen
DK Tøres fladt **FI** Kuivaa tasoisena **FR** Sécher à plat **IS** þurrkað flatt **IT** Asciugare disteso **NL** Plat drogen, niet in de droger. **SE** Torkas liggande **SI** Sušitije poravnano **AE** لا تجفف مقلوب

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CZ

IT

About this product
eakin® support belts are lightweight fabric belts with an adjustable VELCRO® fastener. They may be suitable for those users who require hernia or abdominal support during periods of physical activity such as walking, gardening, lifting, sports, housework etc.

Intended users
 For ostomates, healthcare professionals and caregivers.

Instructions for use

Choosing the correct size
 Size selection should be ideally performed in consultation with a healthcare professional.

Application
 1. Apply the support belt while lying on your back on a firm surface.
 2. Start with the belt behind your back and draw the VELCRO® closures together and close them in front in a position you find comfortable.
 3. Ensure that the belt is fastened for firm but comfortable support.
 4. The support belt should be worn during periods of physical activity. It should not be worn while sleeping or lying down.

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Warnings and cautions
eakin® support belts should not be used while sleeping or lying down. Ileostomates and urostomates should not fasten the support belt too tightly, in case flow is obstructed.

Other information
eakin® accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to **eakin®**'s current recommendations.

A serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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